DuoSwift™ cleaning brush

Écouvillon DuoSwift™
DuoSwift™ Reinigungs- und Lamellenbürste
Spazzolino per pulizia DuoSwift™
Cepillo de limpieza DuoSwift™
Escova de limpeza DuoSwift™
DuoSwift™ rensbørste
DuoSwift™ reinigungsborsteltje
DuoSwift™ temizleme fırçası
DuoSwift™ 세정 브러시
Чистящая щетка DuoSwift™

1 DuoSwift™ cleaning brush & 1 valve/control head brush

Un écouvillon DuoSwift™ et un écouvillon pour vannes/cages à piston
1 DuoSwift™ Reinigungs- und Lamellenbürste und 1
Ventilbürste/1 spazzolino per pulizia DuoSwift™ e 1 spazzolino per pulizia della valvola/unità di controllo
1 cepillo de limpieza DuoSwift™ y 1 cepillo del cabezal de control/válvula
1 escova de limpeza DuoSwift™ e 1 escova para cabeça de controlo/válvula
1 DuoSwift™ rensbørste og 1 ventil-/styrehovedbørste
1 DuoSwift™ reinigungsborsteltje en 1 borsteltje voor kleppen/stuurkoppen
1 DuoSwift™ temizleme fırçası ve 1 valf/kontrol başlığı fırçası
1 DuoSwift™ 세정 브러시 & 1 밸브/컨트롤 헤드 브러시
1 чистящая щетка DuoSwift™ и 1 щетка для клапанов/блока управления

Reorder No. 00711618

Référence de commande 00711618
Nachbestell-Nr. 00711618
N. di riordino 00711618
N.º de pedido 00711618
N.º de encomenda 00711618
Genbestillingsnr. 00711618
Nabestelnr. 00711618
Yeni Sipariş Numarası 00711618
주문번호 00711618
Номер для заказа: 00711618

Reorder No. 00711619

Référence de commande 00711619
Nachbestell-Nr. 00711619
N. di riordino 00711619
N.º de pedido 00711619
N. de encomenda 00711619
Genbestillingsnr. 00711619
Nabestelnr. 00711619
Yeni Sipariş Numarası 00711619
주문번호 00711619
Номер для заказа: 00711619

INSTRUCTIONS FOR USE

MODE D'EMPLOI
GEBRAUCHSANLEITUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
BRUGERVEJLEDNING
GEBRUIKSAANWIJZING
KULLANIM TALIMATLARI
사용설명서
ИНСТРУКЦИИ ПО ПРИМЕНЕНИЮ

These products have been manufactured not to include latex.

- **Product #00711618:** The DuoSwift™ cleaning brush- channel brush & squeegee combo is to only be used in channels **2.8 – 7.0 mm** in diameter.
- **Product #00711619:** The DuoSwift™ cleaning brush (channel brush & squeegee combo) & valve/control head brush. The DuoSwift™ cleaning brush (channel brush & squeegee combo) is to only be used in channels **2.8 – 7.0 mm** in diameter. The valve/control head brush is to only be used in **control head valve openings, the accessory/instrument channel and port.**

Intended Use:

The single use DuoSwift™ cleaning brush is a combination cleaning brush. The flexible plastic sheath has a tapered nylon brush on the leading end and a series of squeegees on the trailing end designed to safely remove debris from flexible endoscopes.

The single use valve/control head cleaning brush is intended to safely remove debris from the valve openings and dials of the control head of flexible endoscopes.

Warnings and Precautions:

- Review Original Equipment Manufacturer's guidelines regarding the care and cleaning of individual endoscope channels prior to the use of any cleaning brush.
- Follow universal precautions, use appropriate infection control principles, and wear appropriate PPE.
- The DuoSwift™ cleaning brush is to be used to clean channels **2.8 – 7.0 mm** in diameter only.
- **Do not use** the DuoSwift™ cleaning brush in channels smaller than **2.8mm**.
- **Always use** the DuoSwift™ cleaning brush in a **wetted scope**.
- **Do not use** the DuoSwift™ cleaning brush in channels in which there is a known retained foreign object (i.e. removed stents, band ligators, other devices, etc.)
- The DuoSwift™ cleaning brush should never be used as a cytology brush.
- **Never insert the squeegee portion of the DuoSwift™ cleaning brush first** (as the leading end) as it will get stuck within the channel.
- Dispose of the DuoSwift™ cleaning brush after each endoscope is cleaned. The mechanical attachment of the brush and squeegee to the plastic sheath cannot be guaranteed to be reliable or secure following one complete endoscope cleaning.
- If sharp bends in the plastic sheath or wire cable are observed during insertion of the brush, the brush should be disposed of to avoid causing damage to the endoscope's operating channels.
- **Never transfer** a used cleaning brush from one endoscope to another endoscope, since cross-contamination could occur and result in patient complications. Disposable cleaning brushes are designed to safely remove debris, some of which can be retained in the bristle structure.
- The DuoSwift™ cleaning brush is intended to clean the suction and accessory channels of flexible endoscopes. **If the flexible endoscope contains an elevator section, clean and brush the elevator and surrounding area according to manufacturer's guidelines.**
- US Endoscopy did not design these devices to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of these devices to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, reesterilizes, and or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.


Contraindications:

- A channel that is smaller than 2.8mm in diameter.
- A channel that is **known to be occluded**.
- **Resistance is met** in attempting to pass the brush through the endoscope.

Directions for Use:

1. Prior to clinical use, open up package and remove device.
2. Familiarize yourself with the device(s) and read all the Instructions for Use.
3. Inspect the package and device for shipping or handling damage, i.e. bent or cracked catheter, missing bristles, brush, or squeegee. If damage is evident **DO NOT USE THESE DEVICES, SAVE THEM FOR RETURN, AND CONTACT YOUR LOCAL PRODUCT SPECIALIST.**
4. **Always use** the DuoSwift™ cleaning brush in a **wetted scope**.
5. DuoSwift™ cleaning brush (Figure 1.):
 - a. Remove valves from the control head of the endoscope.
 - b. Immerse entire endoscope in an appropriately prepared amount of detergent solution.
 - * If cleaning an endoscope with an elevator, brush the elevator and surrounding area according to manufacturer's guidelines.
 - c. Remove all debris from the brush head.
 - d. Never insert the squeegee portion of the DuoSwift™ cleaning brush as the leading end as it will get stuck within the channel.
 - e. Insert the channel brush (leading end) of the DuoSwift™ cleaning brush into the suction cylinder and down the accessory/instrument channel of the insertion tube of the endoscope using short strokes until the channel brush is visibly a few inches outside the distal end.

- f. Remove all debris from the brush head.
 - g. Grasp the channel brush (leading end) of the DuoSwift™ cleaning brush catheter and, using short strokes, pull the squeegee portion through the entire length of accessory channel of the insertion tube of the endoscope until it is completely removed from the endoscope.
 - h. Remove all debris from the squeegee portion.
 - i. Inspect the white catheter, at connections points to brush and squeegee, for cracks, missing bristles/brush/squeegee.
 - j. Repeat steps e-i in order to brush the suction channel of the universal cord and endoscope connector.
 - k. Brush the valve stem openings with the channel brush (leading end) of the DuoSwift™ cleaning brush.
 - l. Remove all debris from the brush head.
 - m. Brush biopsy valve with the channel brush (leading end) of the squeegee cleaning brush.
 - n. Repeat any steps as necessary according to manufacturer's guidelines.
 - o. Dispose of cleaning brush according to facility protocol.
6. If using the valve/control head cleaning brush (Figure 2.):
- a. Brush the suction cylinder housing
 - b. Remove all debris from the brush.
 - c. Brush the control head and control dials.
 - d. Remove all debris from the brush.
 - e. Brush the accessory/instrument channel port of the endoscope.
 - f. Remove all debris from the brush.
 - g. Brush the valve stem openings (Optional if using this brush instead of channel brush for cleaning this area).
 - h. Remove all debris from brush.
 - i. Repeat any steps as necessary according to manufacturer's guidelines.
 - j. Dispose of cleaning brush according to facility protocol.
- * If cleaning an endoscope with an elevator, brush the elevator and surrounding area according to manufacturer's guidelines.

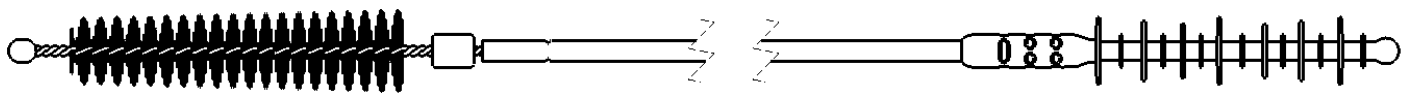Channel brush (leading end)

Squeegee portion (trailing end)

Figure 1.
Squeegee cleaning brush

Figure 2.
Valve/control head brush

Product Disposal:

After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: March 2017

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

US Endoscopy, a wholly owned subsidiary of STERIS Corporation.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy ...and delivering solutions®-Design und alle mit ® oder ™ gekennzeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, to design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, designed US Endoscopy listening...and delivering solutions og alle varemærker markeret med ® eller ™ er registreret i USA's patent- og varemærkeregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 USEndoscopy 가 소유하고 있는 상표입니다.