AquaShield® system CO₂ – Pentax
Système Pentax CO₂ AquaShield®
AquaShield® System CO₂ Pentax
Sistema Pentax CO₂ AquaShield®
Sistema Pentax CO₂ AquaShield®
Sistema Pentax CO₂ AquaShield®
AquaShield® system CO₂ Pentax
AquaShield®-systeem CO₂ Pentax
AquaShield® sistemi CO₂ Pentax
AquaShield® 시스템 CO₂ Pentax

Reorder No. 00711551
Référence de commande 00711551
Nachbestell-Nr. 00711551
N. di riordino 00711551
Nº de pedido 00711551
N.º de encomenda 00711551
Genbestillingsnr. 00711551
Nabestelnr. 00711551
Yeni Sipariş Numarası: 00711551
주문 번호 00711551

INSTRUCTIONS FOR USE

Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso
Instrucciones de uso
Instruções de Utilização
Brugsanvisning
Gebruiksaanwijzing
Kullanım talimatları
사용 설명서

listening...and delivering solutions®

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

This device is not made with natural rubber latex.

INTENDED USE:

The AquaShield® system CO₂ – Pentax is intended to be used with an air or carbon dioxide (CO₂) source from an endoscope with the purpose of supplying sterile water to the endoscope during endoscopic procedures. It is compatible with U.S. commercially available sterile water bottles. This device is intended to be used for not more than 24 hours.

For use with Pentax endoscopes, specifically those endoscopes that have a distal air/water nozzle.

The AquaShield® system CO₂ – Pentax (#00711551) is compatible with commercially available water bottles such as Baxter, UK Baxter, Hospira and BBraun.

WARNINGS AND PRECAUTIONS:

- This disposable medical device is intended to be used for no more than 24 hours after package has been opened. This device should be discarded within this time period regardless of number of uses.
- Never attach or reattach the AquaShield® system CO₂ – Pentax to an endoscope that is/was inside the patient and has not been reprocessed. If multiple endoscopes are used during a procedure and possible contamination is expected on the connections, discard the device after the procedure and before the next patient.
- Monitor the water level in the water bottle during the procedure. Water flow for lens cleaning will stop once the water level drops below the water bottle tubing.
- When using the AquaShield® system CO₂ – Pentax with CO₂ it is recommended an insufflator should be used or a supply tubing with a flow control device to prevent overpressurization of the water bottle.
- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to complications, hazards, and techniques prior to the performance of any endoscopic procedure.
- Use proper aseptic techniques to avoid contamination of the AquaShield® system CO₂ – Pentax during device setup, replacement of water bottles and between patient uses.
- US Endoscopy did not design this device to be reprocessed or used for more than the 24-hour period after the package has been opened, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, reesterilizes, and/or reuses this disposable medical device must bear full responsibility for its safety and effectiveness.

CONTRAINDICATIONS:

Contraindications include those specific to any endoscopic procedure.

INSTRUCTIONS FOR USE:

For Use with Air

1. Prior to clinical use, inspect and familiarize yourself with the device and **review the diagram below**. If there is evidence of damage, **do not use this product and contact your local Product Specialist**.
2. Both *Pinch Clamps* should be in the open position when removed from the package.
3. Open a bottle of sterile water and remove cap. (250cc, 500cc, or 1000cc).
4. Open the sterile pouch and remove the AquaShield® system CO₂ – Pentax. Avoid contact with the *Water Bottle Tubing* to reduce the risk of contamination.
5. Insert the *Water Bottle Tubing* into the sterile water bottle and firmly tighten the threaded *Lid Cap* onto the water bottle. Ensure the cap seats on the bottle without being cocked or cross-threaded. If the lid is cocked or cross-threaded, this can lead to leakage and/or reduced sterile water flow or decreased insufflation.
6. Attach the *Endoscope Connector* to the endoscope.
7. Leave the *Pinch Clamp* on the *Endoscope Tubing* in the open position.
8. Attach the *Air Source Connector* to the light source of the processor.
9. **Close the *Pinch Clamp* on the CO₂ Tubing if air is being used for insufflation and the system is not attached to a CO₂ source/insufflator.**
10. Turn on the light source of the processor and press the Air/Pump button to the on position.
11. **Prime the air/water channel prior to insertion of the endoscope into a patient.** If water pressure or insufflation is low, assess the following:
 - Check to see that the *Lid Cap* is tightly attached to the water bottle (not cocked or cross-threaded).
 - Check that the *Endoscope Connector* is firmly attached to the endoscope.
 - Check that the *Air Source Connector* is firmly attached to the light source of the processor.
 - Remove 1/3 of the sterile water from a full sterile water bottle, this will allow for improved water pressure for better insufflation and water flow.
12. Perform air insufflation and water flow for lens cleaning as described in the Pentax instruction manuals for air/water valve and endoscope being used together with a sterile water bottle.
13. If the water bottle needs to be replaced, use proper aseptic technique.

After Procedure is completed

- Follow the Pentax manufacturer's guidelines for purging the air/water channel system when the procedure is completed.
- **Close the *Pinch Clamp* on the *Endoscope Tubing* and disconnect the *Endoscope Connector* from the endoscope. This will prevent water from leaking from the *Endoscope Connector*.**
- Follow the manufacturer's guidelines for turning off the light source and the Air/Pump of the processor.
- Ensure that the device is discarded within 24 hours after the package has been opened. A label is provided with the device.


For Use with CO₂ (Turn OFF the Air/Pump on the Light Source of the Processor)

1. Prior to clinical use, inspect and familiarize yourself with the device and **review the diagram below**. If there is evidence of damage, **do not use this product and contact your local Product Specialist**.
2. Both Pinch Clamps should be in the open position when removed from the package.
3. Open a bottle of sterile water and remove cap. (250cc, 500cc, or 1000cc).
4. Open the sterile pouch and remove the AquaShield® system CO₂ – Pentax. Avoid contact with the *Water Bottle Tubing* to reduce the risk of contamination.
5. Insert the *Water Bottle Tubing* into the sterile water bottle and firmly tighten the threaded *Lid Cap* onto the water bottle. Ensure the cap seats on the bottle without being cocked or cross-threaded. If the lid is cocked or cross-threaded, this can lead to leakage and/or reduced sterile water flow or decreased insufflation.
6. Connect the *Air Source Connector* to the light source of the processor.
7. Attach the *Endoscope Connector* to the *endoscope*.
8. Leave the *Pinch Clamp* on the *Endoscope Tubing* in the open position.
9. Connect the *CO₂ Connection Port/Luer Lock Connection* to the luer lock connection on the CO₂ insufflator. Turn on the CO₂ source and the insufflator.
10. **Open the CO₂ Tubing Pinch Clamp when CO₂ is being used for insufflation and attach the CO₂ Tubing to a CO₂ source/insufflator.**
11. Follow the manufacturer's guidelines for the use of CO₂ source/insufflator.
12. Turn on the light source of the processor. **Turn off the Air/Pump when using CO₂.**
13. **Prime the air/water channel prior to insertion of the endoscope into a patient.** If water pressure or insufflation is low, assess the following:
 - Check to see that the *Lid Cap* is tightly attached to the water bottle (not cocked or cross-threaded).
 - Check that the *Endoscope Connector* is firmly attached to the endoscope.
 - Check that the *Air Source Connector* is firmly attached to the light source of the processor.
 - Check that the *CO₂ Connection Port/Luer Lock Connection* is firmly attached to the insufflator.
 - Check that the *CO₂ Tubing Pinch Clamp* is in the open position.
 - Remove 1/3 of the sterile water from a full sterile water bottle, this will allow for improved water pressure for better insufflation and water flow.
14. Perform CO₂ insufflation and water flow for lens cleaning as described in the Pentax instruction manuals for air/water valve and the endoscope being used together with a sterile water bottle.
15. If the water bottle needs to be replaced, use proper aseptic technique.

If air is to be resumed, turn off CO₂ source/insufflator and turn On the Air/Pump on light source of the processor. Close the Pinch on the CO₂ tubing.

After Procedure is completed

- Follow the Pentax manufacturer's guidelines for purging the air/water channel system when the procedure is completed.
- **Close the Pinch Clamp on the Endoscope Tubing and disconnect from the endoscope. This will prevent water from leaking from the Endoscope Connector.**
- Follow the manufacturer's guidelines for turning off the light source, the Air/Pump on the processor and CO₂ source/insufflator when the procedure is completed.
- Ensure that the device is discarded within 24 hours after the package has been opened. A label is provided with the device.

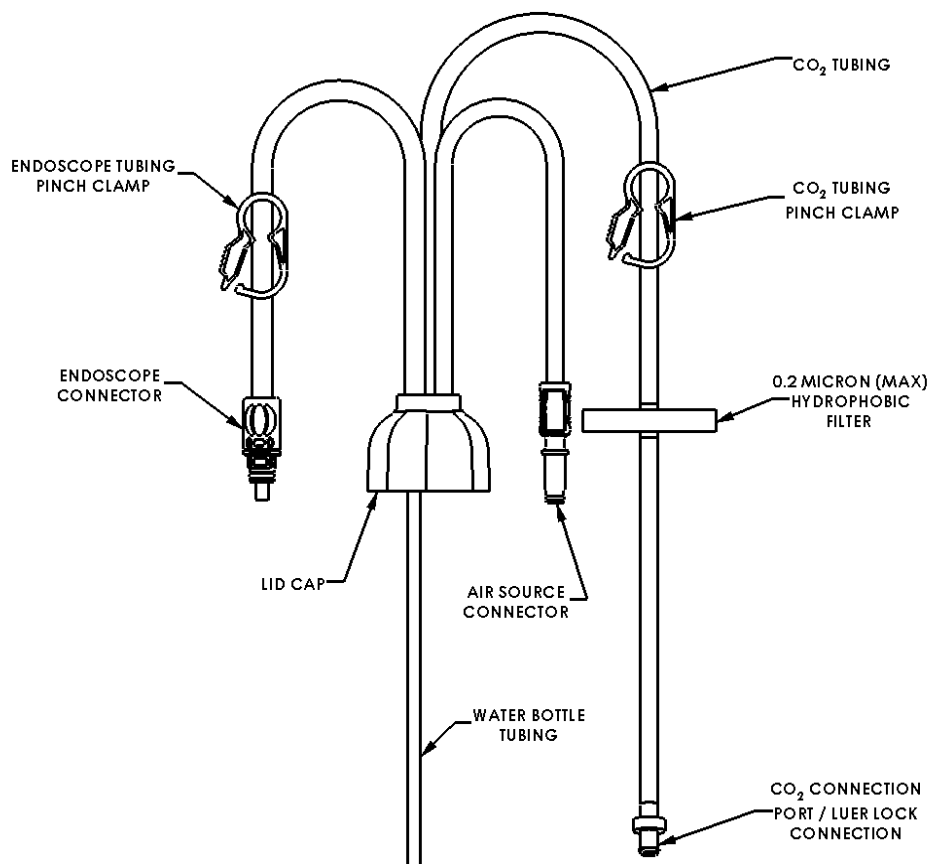Issued Date: October 2016

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

WARNING :

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

US Endoscopy, a wholly owned subsidiary of STERIS Corporation

Pentax® is a registered trademark of Asahi Kogaku Kogyo Kabushiki Kaisha.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont des marques déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions[®], das US Endoscopy listening...and delivering solutions-Design und alle mit © oder [™] bezeichneten Marken sind beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions-designed og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions[®], het ontwerp Listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of [™] zijn gedeponeerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.
Listening...and delivering solutions[®], US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 USEndoscopy 가 소유하고 있는 상표입니다.