AquaShield® system CO₂ – Fujifilm
Système AquaShield® CO₂ – Fujifilm
AquaShield® System CO₂ – Fujifilm
Sistema AquaShield® CO₂ – Fujifilm
Sistema AquaShield® de CO₂ Fujifilm
Sistema de CO₂ AquaShield® – Fujifilm
AquaShield® system CO₂ – Fujifilm
AquaShield®-systeem CO₂ – Fujifilm
AquaShield® sistemi CO₂ – Fujifilm
AquaShield® 시스템 CO₂ – Fujifilm
Система подачи CO₂ AquaShield® Fujifilm
AquaShield® システム CO₂ – Fujifilm

Reorder No. 00711553
N° de commande : 00711553
Nachbestellnr. 00711553
N. di riordino 00711553
N.º de pedido 00711553
N.º de encomenda 00711553
Genbestillingsnr. 00711553
Nabestelnr. 00711553
Yeniden Sipariş No. 00711553
주문 번호 00711553
№ для заказа 00711553
再注文番号 00711553

INSTRUCTIONS FOR USE

MODE D'EMPLOI

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BRUGSANVISNING

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KULLANIM TALİMATLARI

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email global@usendoscopy.com

www.usendoscopy.com

This device is not made with natural rubber latex.

INTENDED USE:

The AquaShield® system CO₂ – Fujifilm is intended to be used with an air or carbon dioxide (CO₂) source from an endoscope with the purpose of supplying sterile water to the endoscope during endoscopic procedures. It is compatible with U.S. commercially available sterile water bottles. This device is intended to be used for not more than 24 hours.

For use with Fujifilm endoscopes, specifically those endoscopes that have a distal air/water nozzle.

The AquaShield® system CO₂ – Fujifilm (#00711553) is compatible with commercially available water bottles such as Baxter, UK Baxter, Hospira and B Braun.

WARNINGS AND PRECAUTIONS:

- This disposable medical device is intended to be used for no more than 24 hours after package has been opened. This device should be discarded within this time period regardless of number of uses.
- Never attach or reattach the AquaShield® system CO₂ – Fujifilm to an endoscope that is/was inside the patient and has not been reprocessed. If multiple endoscopes are used during a procedure and possible contamination is expected on the connections, discard the device after the procedure and before the next patient.
- Monitor the water level in the water bottle during the procedure. Water flow for lens cleaning will stop once the water level drops below the water bottle tubing.
- When using the AquaShield® system CO₂ – Fujifilm with CO₂ it is recommended an insufflator should be used or a supply tubing with a flow control device to prevent overpressurization of the water bottle.
- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to complications, hazards, and techniques prior to the performance of any endoscopic procedure.
- Use proper aseptic techniques to avoid contamination of the AquaShield® system CO₂ – Fujifilm during device setup, replacement of water bottles and between patient uses.
- Do not attempt to reuse, reprocess, refurbish, remanufacture or resterilize this device. US Endoscopy did not design this device nor is it intended to be reused, reprocessed, refurbished, remanufactured, or resterilized. Performing such activities on this disposable medical device presents a safety risk to patients (i.e. compromised device integrity, cross-contamination, infection).

CONTRAINDICATIONS:

Contraindications include those specific to any endoscopic procedure.

INSTRUCTIONS FOR USE:

For Use with Air

1. Prior to clinical use, inspect and familiarize yourself with the device and **review the diagram below**. If there is evidence of damage, **do not use this product and contact your local Product Specialist**.
2. Both *Pinch Clamps* should be in the open position when removed from the package.
3. Open a bottle of sterile water and remove cap. (250cc, 500cc, or 1000cc).
4. Remove 1/3 of the sterile water from the full sterile water bottle. This will allow for improved water pressure for better insufflation and water flow.
5. Open the sterile pouch and remove the AquaShield® system CO₂ – Fujifilm. Avoid contact with the *Water Bottle Tubing* to reduce the risk of contamination.
6. Insert the *Water Bottle Tubing* into the sterile water bottle and firmly tighten the threaded *Lid Cap* to ensure a seal.
7. Attach the *Endoscope Connector* to the endoscope.
8. When using the Fujifilm 530 Series Colonoscopes, attach the ENDO SMARTCAP™ Adapter, part number 100141, to the endoscope in accordance with the manufacturer's instructions for use. Then attach the AquaShield system CO₂ – Fujifilm *Endoscope Connector* to the ENDO SMARTCAP™ Adapter.
9. Leave the *Pinch Clamp* on the *Endoscope Tubing* in the open position.
10. **Close the *Pinch Clamp* on the *CO2 Tubing* if air is being used for insufflation and the system is not attached to a CO₂ source/insufflator.**
11. Turn on the light source of the processor and press the Air/Pump button to the on position.
12. **Prime the air/water channel prior to insertion of the endoscope into a patient.** If water pressure is low, check to see that the *Lid Cap* is tightly closed. Check that the *Endoscope Connector* is firmly attached to the endoscope.
13. Perform air insufflation and water flow for lens cleaning as described in the Fujinon instruction manuals for air/water valve and endoscope being used together with a sterile water bottle.
14. If the water bottle needs to be replaced, use proper aseptic technique.

After Procedure is completed

- Follow the Fujifilm manufacturer's guidelines for purging the air/water channel system when the procedure is completed.
- **Close the *Pinch Clamp* on the *Endoscope Tubing***
- **Ensure that the *Pinch Clamp* is completely closed as evidenced by 2 audible clicks. This will prevent any water from leaking from the *Endoscope Connector*.**
- **Disconnect the *Endoscope Connector* from the endoscope.**
- Follow the manufacturer's guidelines for turning off the light source and the Air/Pump of the processor.
- Ensure that the device is discarded within 24 hours after the package has been opened. A label is provided with the device.

For Use with CO₂ (Turn OFF the Air/Pump on the Light Source of the Processor)

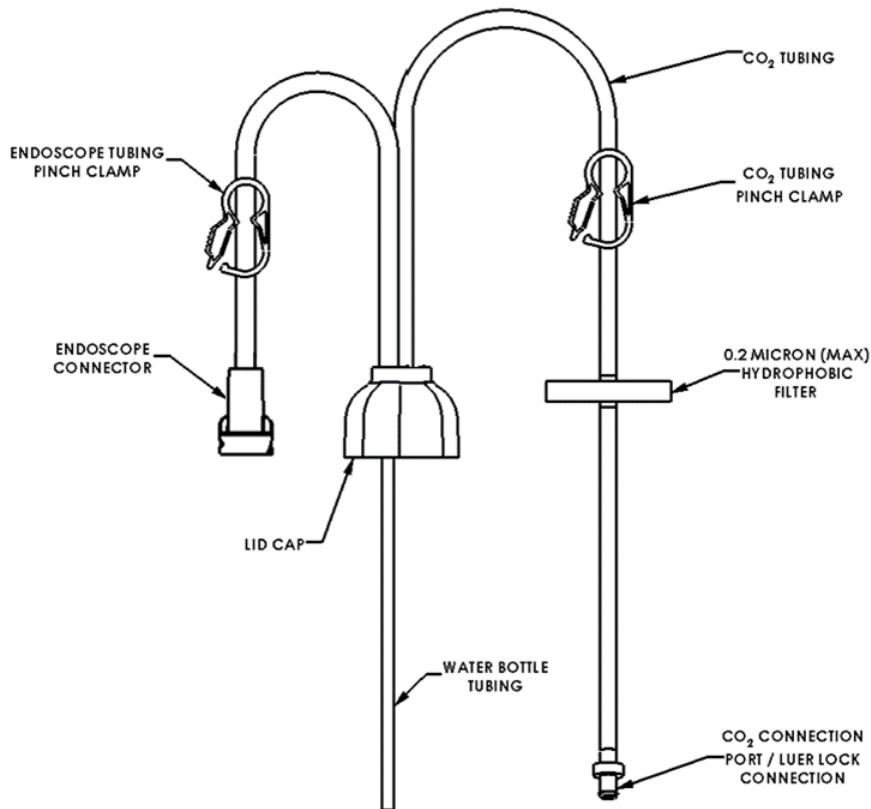
1. Prior to clinical use, inspect and familiarize yourself with the device and **review the diagram below**. If there is evidence of damage, **do not use this product and contact your local Product Specialist**.
2. Both *Pinch Clamps* should be in the open position when removed from the package.
3. Open a bottle of sterile water and remove cap. (250cc, 500cc, or 1000cc).
4. Remove 1/3 of the sterile water from the full sterile water bottle. This will allow for improved water pressure for better insufflation and water flow.
5. Open the sterile pouch and remove the AquaShield® system CO₂ – Fujifilm. Avoid contact with the *Water Bottle Tubing* to reduce the risk of contamination.
6. Insert the *Water Bottle Tubing* into the sterile water bottle and firmly tighten the threaded *Lid Cap* to ensure a seal.
7. Attach the *Endoscope Connector* to the endoscope.

8. When using the Fujifilm 530 Series Colonoscopes, attach the ENDO SMARTCAP™ Adapter, part number 100141, to the endoscope in accordance with the manufacturer's instructions for use. Then attach the AquaShield system CO₂ – Fujifilm *Endoscope Connector* to the ENDO SMARTCAP™ Adapter.
9. Leave the *Pinch Clamp* on the *Endoscope Tubing* in the open position.
10. Connect the *CO₂ Connection Port/Luer Lock Connection* to the luer lock connection on the CO₂ insufflator. Turn on the CO₂ source and the insufflator.
11. **Open the Pinch Clamp on the CO₂ tubing when CO₂ is being used for insufflation and the tubing is attached to a CO₂ source/insufflator.**
12. Follow the manufacturer's guidelines for the use of CO₂ source/insufflator.
13. Turn on the light source of the processor. **Turn off the Air/Pump when using CO₂.**
14. **Prime the air/water channel prior to insertion of the endoscope into a patient.** If water pressure is low, check to see that the *Lid Cap* is tightly closed. Ensure that all connections are secure.
15. Perform CO₂ insufflation and water flow for lens cleaning as described in the Fujinon instruction manuals for air/water valve and the endoscope being used together with a sterile water bottle.
16. If the water bottle needs to be replaced, use proper aseptic technique.

If air is to be resumed, turn off CO₂ source/insufflator and turn On the Air/Pump on light source of the processor. Close the Pinch on the CO₂ tubing.

After Procedure is completed

- Follow the Fujinon manufacturer's guidelines for purging the air/water channel system when the procedure is completed
- **Close the Pinch Clamp on the Endoscope Tubing.**
- **Ensure that the Pinch Clamp is completely closed as evidenced by 2 audible clicks. This will prevent any water from leaking from the Endoscope Connector**
- **Disconnect the Endoscope Connector from the endoscope.**
- Follow the manufacturer's guidelines for turning off the light source, the Air/Pump on the processor and CO₂ source/insufflator when the procedure is completed.
- Ensure that the device is discarded within 24 hours after the package has been opened. A label is provided with the device.

Issued Date: January 2018

Product Disposal:
 After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

WARNING:
 An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

US Endoscopy, a wholly owned subsidiary of STERIS Corporation

Fujifilm® is a registered trademark of FujiFilm Corporation.

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Listening...and delivering solutions®, дизайн US Endoscopy listening...and delivering solutions и все товарные знаки, отмеченные обозначениями © или ™, зарегистрированы в Бюро патентов и торговых знаков США либо являются товарными знаками, принадлежащими компании.
Listening...and delivering solutions[®], le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont des marques déposées auprès de l'U.S. Patent and Trademark Office (Bureau américain des brevets et des marques de commerce) ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy listening...and delivering solutions-Design und alle mit ® oder ™ bezeichneten Marken sind beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragene Marken oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions[®], US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions[®], het ontwerp van Listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn gedeponeerd bij het Amerikaanse Octrooibureau (Patent and Trademark Office) of zijn handelsmerken van US Endoscopy.