Revital-Ox

Bedside Concentrate

Revital-Ox Bedside Concentrate is an enzymatic detergent and infused sponge formulated to pre-clean and remove protein, carbohydrates and lipid soils from the exterior and interior of flexible or rigid endoscopes and surgical instruments.

INSTRUCTIONS FOR USE: For Endoscope Bedside Cleaning

1. Tear open the package at the notch
2. Add warm tap water to bring volume to the designated fill line
3. While still in the pouch, lightly massage the sponge to mix the enzymatic concentrate with the water
4. Squeeze excess enzymatic detergent from the sponge
5. Remove the sponge from the pouch and wipe down the outside of the endoscope
6. Place the distal end of the endoscope into the pouch and suction the remaining enzymatic detergent through the biopsy suction channel
7. Cycle air and the detergent through the channel
8. Purge channel with clear water and rinse the endoscope thoroughly
9. Place the endoscope into the transport container and send to reprocessing area
10. Keep endoscope moist until final cleaning

REORDER# 2D86QZ

Discard packaging and sponge immediately after use
FOR SINGLE USE ONLY

Contents: 1 Sponge with
2 ml. (16.9 fl. oz.) of
Enzymatic Detergent

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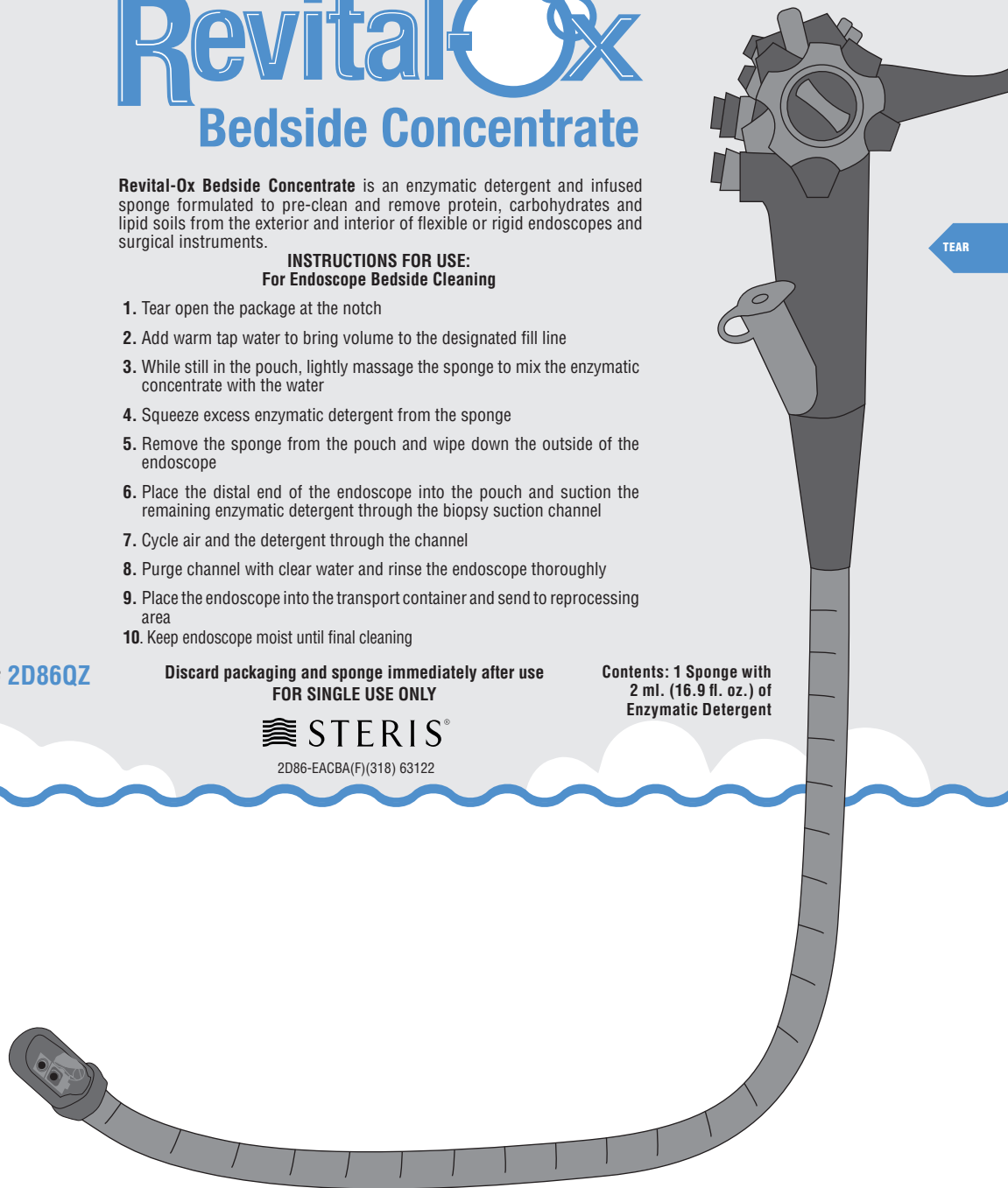
For Surgical Instrument Application

1. Follow Steps 1-4 on front of package
2. Remove sponge from the pouch and wipe down the outside of the device
3. Rinse thoroughly

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DANGER

CONTAINS: Subtilisins (proteolytic enzymes) (CAS # 9014-01-1) (0.1-1.0%), Alcohols, C12-14-secondary, ethoxylated surfactants (CAS# 84133-50-6) (1-5%), 1-Octanamine, N,N-dimethyl-N-oxide (CAS# 2605-78-9) (1-3%). Causes skin irritation. Causes eye irritation. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. Avoid breathing mist, spray, vapours. Wash hands thoroughly after handling. Use only in well-ventilated area. Wear eye protection, protective clothing, protective gloves. In case of inadequate ventilation, wear respiratory protection. **IF ON SKIN**: Wash with plenty of soap and water. **IF INHALED**: Remove person to fresh air and keep at rest in a position comfortable for breathing. **IF IN EYES**: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical attention. If eye irritation persists: Get medical attention. If experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician. Take off contaminated clothing and wash it before reuse. For hospital and professional use only. Not for home use. See Safety Data Sheet for additional information.

ADD WATER TO FILL LINE

REORDER#
2D86QZ

Manufactured by:
STERIS Corporation
7501 Page Avenue
St. Louis, Missouri 63133 • USA
800-548-4873 • www.steris.com
Product Made in U.S.A.

RevitalOx
Bedside Concentrate

STERIS[®]

2D86-EACBA(B)(318) 63123

Distributed by:
STERIS Canada Sales ULC
375 Britannia Road East.
Mississauga, Ontario L4Z 3E2
Canada • 800-661-3937
Produit fabriqué aux États-Unis.