


## Assembly Instructions for Genii Accessories

### Mounting Accessories include:

- 1 ea. Shelf with pre-mounted hardware
- 1 ea. Extrusion with pre-mounted hardware
- 1 ea. Base with casters
- 1 ea. Large Basket with pre-mounted hardware
- 1 ea. Small Basket with pre-mounted hardware
- 1 ea. Rear Handle with pre-mounted hardware
- 1 ea. Column Cap
- 1 ea. 4mm Hex Key
- 1 ea. 5mm Hex Key
- 1 ea. TX30 Hex Key

1. Unwrap and verify that all parts listed above are included.

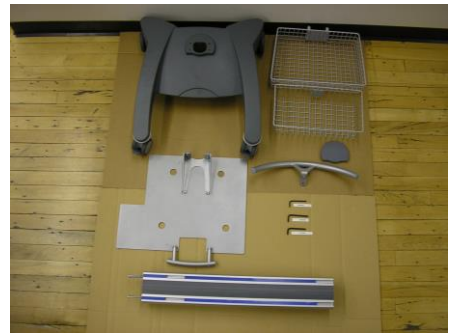

## Assembly Instructions for Genii Accessories

2. Find a solid work surface, either table or floor. Remove the pre-mounted hardware from the extrusion and set aside. Place base on top of extrusion. Using TX30 Hex Key provided and the pre-mounted hardware that was set aside, begin to screw the base onto the extrusion.



3. Next remove only the long T Channel cover strips (blue strips) pre-installed.

Note: The extrusion column has two T Channel cover strips (blue strips) pre-installed. Remove only the long T Channel covers and set aside.

**DO NOT** remove the short T Channel (blue strips).



## Assembly Instructions for Genii Accessories

4. Next install large basket. Align the pre-mounted hardware into the front of the column slots. See photo. Slide the basket down until it rests on the short T Channel cover (blue strip). Tighten screws using hex key provided.



5. Lay cart on its back and press the long T Channel cover strips (blue strips, long) into the extrusion so it rests on the mounted basket.




## Assembly Instructions for Genii Accessories

6. Next install Shelf. Align the pre-mounted hardware into the front of the column slots and tighten using hex key provided. See photo.



## Assembly Instructions for Genii Accessories

7. Next install small basket on the back of the extrusion. Remove top plastic strip and align the pre-mounted hardware into the back of the column slot, slide down on top of bottom strip and tighten using hex key provided. See photo.

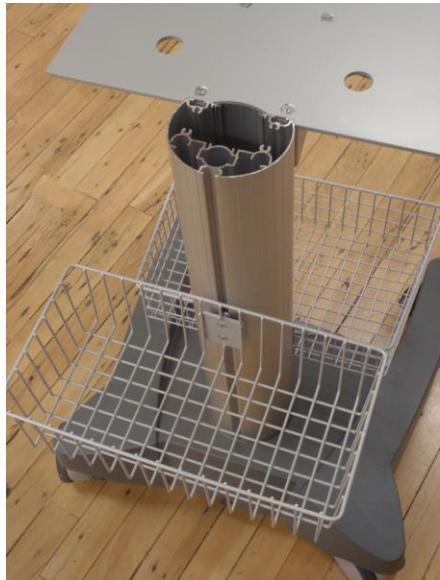



## Assembly Instructions for Genii Accessories

8. Replace plastic strip.



9. Next install handle. Align the pre-mounted hardware into the back of the column slots and tighten using hex key provided. See photo.



## Assembly Instructions for Genii Accessories

10. Properly orient the column cover over the mounting column and press into place.

This completes the assembly process

