Traxtion™ hexagonal snare

Anse hexagonale Traxtion™
Traxtion™ hexagonale Schlinge
Traxtion™ - ansa esagonale
Traxtion™ - lazo hexagonal
Ansa hexagonal Traxtion™
Traxtion™ heksagonal slyng
Traxtion™ zeshoekige poliepsnoerder
Traxtion™ altigen snare
Traxtion™ 6각형 올가미

Short Throw™ snare – mini-oval

Anse Short Throw™ - mini-ovale
Short Throw™ Schlinge – Mini, oval

Short Throw™ - ansa – mini - ovale
Short Throw™ - lazo – mini-ovalado
Ansa Short Throw™ – mini-oval
Short Throw™ slyng – mini-oval
Short Throw™ poliepsnoerder – mini-ovaal
Short Throw™ snare - mini-oval
Short Throw™ 올가미 - 소형 타원

Short Throw™ - snare – oval

Short Throw™ - anse – ovale
Short Throw™ Schlinge – oval
Short Throw™ - ansa – ovale
Short Throw™ - lazo – ovalado
Ansa Short Throw™ – oval
Short Throw™ slyng – oval
Short Throw™ - poliepsnoerder – ovaal
Short Throw™ - snare – oval
Short Throw™ - 올가미 - 타원

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Exacto™ soğuk snare
Exacto™ 콜드형 올가미

Short Throw™ - snare – enteroscope

Short Throw™ - anse - entéroscope
Short Throw™ Schlinge – Enteroskop
Short Throw™ - ansa – enteroscopio
Short Throw™ - lazo – enteroscopia
Ansa Short Throw™ – enteroscópio
Short Throw™ slyng – enteroskop
Short Throw™ - poliepsnoerder – enteroscoop
Short Throw™ - snare – enteroskop
Short Throw™ - 올가미 – 장경

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Intended Use:

The Rotator™ snares, Short Throw™ snares and Traxtion™ hexagonal snare are intended to be used with diathermic energy for the endoscopic resection of polyps in the gastrointestinal tract. The Exacto™ cold snare is intended to be used without diathermic energy for the endoscopic resection of diminutive polyps in the gastrointestinal tract.

Warnings and Precautions:

- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to techniques, contraindications, complications and hazards prior to any endoscopic procedure.
- Do not use any of these devices if there is any apparent shipping or handling damage.
- These devices are compatible with an endoscope channel of 2.8mm or larger.
- Endoscopic polypectomy with diathermic snares should not be performed without a thorough understanding of the principles of diathermic energy. Consult the electrosurgical generator manufacturer's instruction booklet for proper settings and use of the generator. In order to ensure that the insulating properties of the diathermic snares are not compromised, do not exceed the maximum rated peak recurring voltage stated for each high frequency mode (see chart below).

High Frequency Mode	Maximum Rated Recurring Peak Voltage
Cut	1200 Volts
Coagulation Burst	2100 Volts
Coagulation Spray	5200 Volts

- US endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. US Endoscopy does not take responsibility for the safety and effectiveness of these disposable medical devices following reprocessing, remanufacturing, and/or resterilization by any institution, practitioner or third party.

Directions for Use:

1. Following suggested medical techniques, prepare the patient for endoscopy.
2. **For diathermic snares:** Attach the respective Olympus style active cord to the snare's diathermic handle connection. Note that the Exacto™ cold snare is not intended to be used with diathermic energy and, therefore, has no diathermic handle connection.
3. Ensure the snare loop is fully retracted into the sheath prior to insertion into the endoscope.
4. **For Rotator™ snare only:** Rotation may be accomplished by twisting the handle in the direction of the desired movement. Rotation is best achieved with the loop in the center of the lumen and not lying against the gastrointestinal wall.
5. When the polyp has been endoscopically visualized, resect the polyp using proper surgical technique. **The electrosurgical generator should be set to the proper setting that will allow resection with controlled hemostasis.**
6. Once the polyp(s) has been satisfactorily resected, the polyp fragment(s) should be removed and the specimen(s) prepared according to standard technique for histologic evaluation.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: April 2010

Warning:

An issued or revision date for these instructions is included for the user's information. In the event two years has elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

Olympus® is a registered trademark of Olympus Corp.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent und Trademark Office (US-Patent- und Markenamt) registriert wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, el US Endoscopy listening...and delivering solutions design, y todas las marcas que lleven ® o ™ están registradas en la oficina U.S. Patent and Trademark, o son marcas comerciales de US Endoscopy.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registados no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, het ontwerp 'listening...and delivering solutions' van US Endoscopy en alle aanduidingen met ® of ™ zijn geregistreerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.