## Boost® head positioner

Positionneur de tête Boost®  
Boost®-Kopfstütze  
Poggiatesta Boost®  
Dispositivo de control de posición de la cabeza Boost®  
Posicionador de cabeça Boost®  
Boost® hovedholder  
Boost® -hoofdsteu  
Boost® baş konumlayıcı  
Boost® 헤드 포지셔너

## Reorder No. 00711654

Référence de commande 00711654  
Nachbestell-Nr. 00711654  
N. di riordino 00711654  
N° de pedido 00711654  
N.º de encomenda 00711654  
Genbestillingsnr. 00711654  
Nabestelnr. 00711654  
Yeni Sipariş Numarası: 00711654  
주문 번호 00711654

---

## INSTRUCTIONS FOR USE

MODE D'EMPLOI  
GEBRAUCHSANLEITUNG  
ISTRUZIONI PER L'USO  
INSTRUCCIONES DE USO  
INSTRUÇÕES DE UTILIZAÇÃO  
BRUGSANVISNING  
GEBRUIKSAANWIJZING  
KULLANIM TALİMATLARI  
사용 설명서

---

*listening... and delivering solutions\**

5976 Heisley Road  
Mentor, OH 44060 USA

*phone* +1 440 / 639.4494

*fax* +1 440 / 639.4495

*email* [global@usendoscopy.com](mailto:global@usendoscopy.com)

[www.usendoscopy.com](http://www.usendoscopy.com)

**NON-STERILE**

**Rx Only (U.S.A.)**

**Intended Use:**


The single use **Boost® head positioner** is intended to stabilize a patient's head position during endoscopy procedures and cradle the head in a position for endoscopic access to the mouth.

**Warnings and Precautions:**

- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to complications, hazards and techniques prior to the performance of any endoscopic procedure.
- Inspect the package for shipping or handling damage. If damage is evident, do not use this device.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- This disposable medical device is not intended for reuse. Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses this disposable medical device must bear full responsibility for its safety and effectiveness.
- User is responsible for the application and use of appropriate and accepted positioning techniques with the device.

**Directions for Use:**

1. Prior to clinical use, familiarize yourself with the device.
  - Read the "Warnings and Precautions" and the "Directions for Use."
2. Carefully place the head positioner under the patient's head prior to the insertion of the endoscope (See Fig. 1).

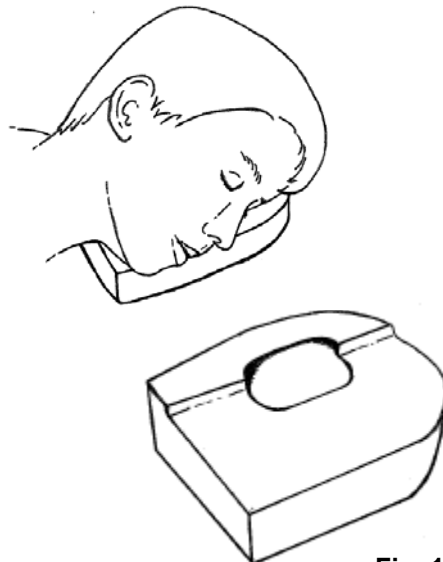**Fig. 1**

**Product Disposal:**

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**Issued Date:** August 2010

**Warning:**

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

**Listening...and delivering solutions®**, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

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**Listening...and delivering solutions®**, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
**Listening...and delivering solutions®**, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
**Listening...and delivering solutions®**, US Endoscopy listening...and delivering solutions design e tutti i marchi contrassegnati da ® o ™ sono stati registrati presso l'US Patent and Trademark Office degli Stati Uniti o sono marchi di fabbrica di proprietà di US Endoscopy.
**Listening...and delivering solutions®**, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
**Listening...and delivering solutions®**, US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
**Listening...and delivering solutions®**, het ontwerp listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn gedeponeerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.