## Cleaning Brush

Cleaning Brush – channel  
 Écouvillon - pour canal  
 Reinigungsbürste – Kanal  
 Spazzolino per pulizia dei canali  
 Cepillo de limpieza – canal  
 Escova de limpeza – canal  
 Rensebørste – kanal  
 Reinigingsborsteltje – kanaal  
 Temizleme Fırçası – kanal  
 세정 브러시 – 채널

Reorder No. 00711601

Cleaning Brush – channel – enteroscope  
 Écouvillon - pour canal - entéroscopie  
 Reinigungsbürste – Kanal – Enteroskop  
 Spazzolino per pulizia dei canali - enteroscopio  
 Cepillo de limpieza – canal – enteroscopio  
 Escova de limpeza – canal – enteroscópio  
 Rensebørste – kanal – enteroskop  
 Reinigingsborsteltje – kanaal – enteroscoop  
 Temizleme Fırçası - kanal - enteroskop  
 세정 브러시 - 채널 - 장경

Reorder No. 00711606

Cleaning Brush – channel – maxi  
 Écouvillon - pour canal - maxi  
 Reinigungsbürste – Kanal – Maxi  
 Spazzolino per pulizia dei canali - maxi  
 Cepillo de limpieza – canal – maxi  
 Escova de limpeza – canal – maxi  
 Rensebørste – kanal – maxi  
 Reinigingsborsteltje – kanaal – maxi  
 Temizleme Fırçası - kanal - maksı  
 세정 브러시 - 채널 - 맥시

Reorder No. 00711603

Cleaning Brush – control head  
 Brosse – cage a piston  
 Reinigungsbürste – Steuerungskopf  
 Spazzolino per pulizia dell'unità di controllo  
 Cepillo de limpieza – cabezal de control  
 Escova de limpeza – cabeça de controle  
 Rensebørste – styrehoved  
 Reinigingsborsteltje – stuurkop  
 Temizleme Fırçası - kontrol başlığı  
 세정 브러시 - 컨트롤 헤드

Reorder No. 00711610  
 Reorder No. 00711611

## Double Header™ - channel and control head combo

Double Header™ - combinaison de brosses pour canal et cage a piston  
 Double Header™ - Kanal- und Steuerungskopfkombination  
 Double Header™ - spazzolino a testine intercambiabili per pulizia dei canali e dell'unità di controllo  
 Double Header™ - combinación de canal y cabezal de control  
 Double-Header™ - combinado de canal e cabeça de controle  
 Double Header™ - kombination af kanal og styrehoved  
 Double Header™ - kanaal- en stuurkopcombinatie  
 Double Header™ - kanal ve kontrol başlığı birlikte  
 Double-Header™ - 채널 및 컨트롤 헤드 콤보

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NON-STERILE

Rx Only (U.S.A.)

- **Product #00711601, #00711607 and #00711612**– Use only in channels 2.0mm and larger
- **Product #00711603 and #00711606** – Use only in channels 2.8mm and larger
- **Product #00711609** – Use only in channels 2.0mm and larger and in control head valve openings
- **Product #00711610 and #00711611** – Use only in control head valve openings
- **Product #00711616 and #00711617** – Use only in channels 1.0mm and larger

#### Intended Use:

Based on the product number, the cleaning brush(es) are used for endoscope accessory/suction channel, auxiliary water channel, balloon inflation channel (EUS endoscope) or control head cleaning. The intended use of these cleaning brushes is as follows:

- The channel cleaning brush is a nylon Flexi Brush™ attached to a flexible plastic sheath designed to safely remove debris from flexible endoscopes.
- The control head cleaning brush is used to safely remove debris from the valve openings and dials of the control head of flexible endoscopes.
- The Small Wonder - cleaning brush is designed to safely remove debris from 1.0mm and larger channels, such as the auxiliary water channel of some flexible endoscopes, the balloon inflation channel (EUS endoscopes), or flexible bronchoscope accessory channels.

#### Warnings and Precautions:

- The cleaning brush should never be used as a cytology brush.
- Original Equipment Manufacturer's guidelines regarding the care and cleaning of individual endoscope channels must be reviewed prior to the use of any cleaning brush.
- Dispose of the cleaning brush after initial use. The mechanical attachment of the brush(es) to the plastic sheath cannot be guaranteed to be reliable or secure following initial use.
- If sharp bends in the plastic sheath or wire cable are observed during insertion of the brush, the brush should be disposed of to avoid causing damage to the endoscope's operating channels.
- **Never transfer** a used cleaning brush from one endoscope to another endoscope, since cross-contamination could occur and result in patient complications. Disposable cleaning brushes are designed to safely remove debris, some of which can be retained in the bristle structure.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- These products are intended for single patient use only. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, reesterilizes, and/or reuses these disposable devices must bear full responsibility for their safety and effectiveness.

#### Contraindications:

- If the channel(s) are known to be occluded or resistance is met in attempting to pass the brush through the endoscope, **do not force** the cleaning brush through the endoscope, as this may result in damage to the channel.
- Do not use a specific cleaning brush in a specific channel if use is not recommended by the endoscope's Original Equipment Manufacturer.

#### Directions for use:

1. Channel cleaning brush:
  - a) Using the cleaning media of choice, advance the cleaning brush through the **entire length of the endoscope before reversing direction**. Reversing direction inside the endoscope could cause the brush to expand, which could potentially damage the endoscope.
  - b) Using the cleaning media of choice, pass the channel cleaning brush through the valve stem openings.
2. Control head cleaning brush:
  - a) Remove valve(s) from the control head of the endoscope. Using the cleaning media of choice, use the brush to gently cleanse the valve openings. Avoid twisting or torquing the brush handle. The brush handle will prevent the brush from being pushed too deeply into the valve openings.
  - b) Using the cleaning media of choice, clean control head and control head dials.

#### Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**Issued Date:** April 2010

#### Warning:

An issued or revision date for these instructions is included for the user's information. In the event two years has elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent und Trademark Office (US-Patent- und Markenamt) registriert wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions® , o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i U.S. Patent and Trademark Office eller er varemærker, der tilhører US Endoscopy
Listening...and delivering solutions®, het ontwerp listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn geregistreerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청 (U.S. Patent and Trademark Office) 에 등록되어 있거나 US Endoscopy 가 소유하고 있는 상표입니다.