Cytology Brush – bronchoscope

Brosse cytologique - bronchoscopie
Zytologiebürste - Bronchoskop
Spazzolino per citologia – broncoscopio
Cepillo de citología - bronquioscopia
Escova de citologia – broncoscópico
Cytologibørste – bronkoskop
Cytologieborstel – bronchoscoop
Sitoloji Fırçası - bronkoskop

Reorder No. 00711401

Cytology Brush – gastroscope

Brosse cytologique - gastroscopie
Zytologiebürste - Gastroskop
Spazzolino per citologia – gastroscopio
Cepillo de citología - gastroscopia
Escova de citologia – gastroscópico
Cytologibørste – gastroskop
Cytologieborstel – gastroscop
Sitoloji Fırçası - gastroskop

Reorder No. 00711402

Cytology Brush – colonoscope

Brosse cytologique - coloscopie
Zytologiebürste - Kolonoskop
Spazzolino per citologia – colonscopio
Cepillo de citología - colonoscopia
Escova de citologia – colonoscópico
Cytologibørste – kolonoskop
Cytologieborstel – colonoscoop
Sitoloji Fırçası - kolonoskop

Reorder No. 00711403

Cytology Brush – enteroscope

Brosse cytologique - entéropscopie
Zytologiebürste - Enteroskop
Spazzolino per citologia – enteroscopio
Cepillo de citología - enteroscopia
Escova de citologia – enteroscópico
Cytologibørste – enteroskop
Cytologieborstel – enteroscoop
Sitoloji Fırçası - enteroskop

Reorder No. 00711405

INSTRUCTIONS FOR USE

Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso
Instrucciones de uso
Instruções de Utilização
Brugsanvisning
Gebruiksaanwijzing
Kullanım talimatları

listening...and delivering solutions®

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

STERILE EO

Rx Only (U.S.A.)

Intended Use:

The disposable **Cytology Brush** is designed to collect cells from the mucosa of the GI tract or bronchus.

Warnings and Precautions:

- Flexible Endoscopy procedures should only be performed by persons having adequate training and familiarity with endoscopic procedures. Consult the medical literature relative to techniques, complications and hazards prior to the performance of any endoscopic procedure.
- Excessive brushing may cause hemorrhage.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- The disposable **cytology brush** product line is intended for single patient use only. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses these disposable devices must bear full responsibility for their safety and effectiveness.

Directions for Use:

1. **Following the suggested medical technique, prepare the patient for Endoscopy.**
2. Remove the brush from the sterile package, retract into the sheath and insert it through the endoscope. Once the targeted area has been endoscopically identified, advance the brush back and forth a few times over the area. Upon completion of brushing, retract the brush into the sheath before removing it from the endoscope in order to preserve the harvest of cells. Endoscopically confirm the retraction of the brush. Remove the brush assembly from the endoscope following completion of the procedure. Properly fix the specimen for cytologic evaluation.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: April 2010

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

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Made in the U.S.A.