## Articulator™ - injection needle

Articulator™ – Aiguille à injection  
Articulator™ – Injektionsnadel  
Articulator™ – ago per iniezione  
Articulator™ – Aguja de inyección  
Articulator™ – agulha de injeção  
Articulator™ – injektionsnål  
Articulator™ – injectienaald  
Articulator™ – enjeksiyon iğnesi  
Articulator™ – 주사기

**Reorder No. 00711803** (4mm / 160cm / 25ga)  
**Reorder No. 00711804** (5mm / 160cm / 25ga)  
**Reorder No. 00711807** (5mm / 230cm / 25ga)  
**Reorder No. 00711808** (5mm / 350cm / 25ga)

## Carr-Locke - injection needle

Carr-Locke – Aiguille à injection  
Carr-Locke – Injektionsnadel  
Carr-Locke – ago per iniezione  
Carr-Locke – Aguja de inyección  
Carr-Locke – agulha de injeção  
Carr-Locke – injektionsnål  
Carr-Locke – injectienaald  
Carr-Locke – enjeksiyon iğnesi  
Carr-Locke – 주사기

**Reorder No. 00711811** (5mm / 230cm / 25ga)  
**Reorder No. 00711813** (5mm / 230cm / 25ga)

## Vari-Safe™ - injection needle

Vari-Safe™ – Aiguille à injection  
Vari-Safe™ – Injektionsnadel  
Vari-Safe™ – ago per iniezione  
Vari-Safe™ – Aguja de inyección  
Vari-Safe™ – agulha de injeção  
Vari-Safe™ – injektionsnål  
Vari-Safe™ – injectienaald  
Vari-Safe™ – enjeksiyon iğnesi  
Vari-Safe™ – 주사기

**Reorder No. 00711819** (5mm / 230cm, 23ga)

---

## INSTRUCTIONS FOR USE

Instructions d'utilisation  
Gebrauchsanweisung  
Istruzioni per l'uso  
Instrucciones de uso  
Instruções de Utilização  
Brugsanvisning  
Gebruiksaanwijzing  
Kullanım talimatları  
사용 설명서

---

Rx Only (U.S.A.)

### Intended Use:

These single use, disposable, flexible sheath injection needles are intended to be used for the injection of various types of media through flexible endoscopes.

### Warnings and Precautions:

- Flexible endoscopy procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques. Consult the medical literature relative to techniques, contraindications, complications and hazards prior to any endoscopic procedure. If reference information for endoscopic technique or contraindications is needed, call US Endoscopy.
- Do not use an injection needle if there is any apparent shipping or handling damage.
- These injection needles are compatible with endoscope channels of 2.8mm or larger.
- **For Carr-Locke injection needles only:** Do not attempt to actuate the injection needle with the catheter in a straightened position. The injection needle has been engineered and manufactured with a specific "preload" that is necessary to assure a full projection and full retraction of the needle from the distal hub when the device is in very tortuous configurations. If the injection needle is actuated with the catheter in a straightened position, the "preload" may cause the needle's inner catheter to collapse and not allow for adequate injection. When handling the injection needle, always be aware of the distal tip of the needle, and visually confirm that the needle is fully retracted into the device.
- **For Vari-Safe™ injection needles only:** Do not attempt to activate or test the injection needle outside of the endoscope if the device is coiled more than once. It is possible that this may cause the injection needle to become lodged proximal to the distal tip spring.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- These injection needles are single use, disposable medical devices; they are not intended for reuse. Any institution, practitioner or third party who reprocesses, refurbishes, remanufactures, reesterilizes, and/or reuses these disposable devices must bear full responsibility for their safety and effectiveness.
- **All of the injection needles included in this IFU (Instruction for Use) are compatible with the following medicinal substances and solutions: Sodium Morrhuate, Sodium Tetracycline Sulfate, Ethynolamine Oleate, Epinephrine, Botox, Polidocanol, Ethanol, Sotradecol, Absolute Alcohol, India Ink, Hypertonic Saline and Staining Media.**
- **In addition, the Vari-Safe™ injection needle (product # 00711819) has been confirmed to be compatible with Triamcinolone (Kenalog). The compatibility of other substances has not been verified and therefore should not be attempted.**
- **The Carr-Locke and Articulator™ injection needles (product numbers 00711811, 00711803, 00711804, 00711807, and 0071808) are not compatible with the Pentax Model 90 series High Definition gastroscopes, while the Carr-Locke injection needle (product number 00711813) is compatible with Pentax Model 90 series High Definition gastroscopes.**

**Contraindications:** The use of this device is contraindicated for uncooperative patients and/or patients with severe coagulopathy.

### Directions for Use:

1. (a) **For Articulator™ & Carr-Locke Injection Needles Only:** Carefully remove the tip protector from the distal tip of the needle. Actuate the proximal luer back and forth, visually observing the needle being adequately advanced from the device's distal end. The spring-loaded handle assists in needle retraction; however, always visually check the retraction as well. **(Note: Product #00711808 version used for enteroscopy does not have this spring-loaded handle feature).**  
 (b) **For Carr-Locke Injection Needle Usage in a Duodenoscope:** When using this needle in a duodenoscope, before actuating the elevator to the full position, assure that the needle sheath is projected adequately past the end of the scope to prevent damage to the needle which, in the retracted position, is positioned inside the spring, proximal to the distal tip.  
 (c) **For Vari Safe™ injection needles only: The needle has been shipped in a "needle projected" position.** This is necessary to keep the distal needle from catching on the distal spring. Carefully remove the tip protector from the distal tip of the needle. Uncoil the needle sheath and ensure that the needle is properly projected. Unlock the luer lock fitting and actuate the proximal luer back and forth, visually observing the needle being adequately advanced from the distal end. The return spring assists in the needle retraction; however, always visually check the retraction as well.  
**Note: Do not stretch the device because stretching may cause the needle projection length to be altered.**
2. To lock the needle in the out position, gently press the proximal luer until it meets the handle's luer, and then slowly twist the proximal luer clockwise until the two luers are fully engaged.
3. Using accepted medical methods, purge the needle assembly of air with the injection media.
4. Ensure that the needle is fully retracted prior to inserting it into the endoscope.
5. Proceed with injection of appropriate media using approved medical techniques until desired results are accomplished.
6. When moving from one injection site to the next, it is advisable to retract the needle into the sheath to protect the mucosa, as well as the endoscope, from trauma/damage.
7. When removing the injection needle from the endoscope, fully retract the needle into the sheath to protect the patient and clinician from injury, and to protect the endoscope from potential damage.

### Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**Issued Date:** April 2010

### Warning:

An issued or revision date for these instructions is included for the user's information. In the event two years has elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

This product and/or its use is protected under United States patent number 6,666,847.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques de commerce détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy listening...and delivering solutions-Design und alle mit ® oder ™ bezeichneten Marken sind beim U.S. Patent und Trademark Office (US-Patent- und Markenamt) eingetragene Marken oder Warenzeichen, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.