Ponsky Endo-Sock™ - retrieval pouch

Ponsky Endo-Sock™ - poche de recuperation
Ponsky Endo-Sock™-Entnahmebeutel
Ponsky Endo-Sock™ cestello per asportazione
Bolsa de recuperación Ponsky Endo-Sock™
Bolsa de recolha Ponsky Endo-Sock™
Ponsky Endo-Sock™ opsamlingspose
Ponsky Endo-Sock™ verwijderzakje
Ponsky Endo-Sock™ çıkarma kesesi
Ponsky Endo-Sock™ 적출 주머니

Reorder No. 00711042

Référence de commande 00711042
Nachbestell-Nr. 00711042
N. di riordino 00711042
Nº de pedido 00711042
N.º de encomenda 00711042
Genbestillingsnr. 00711042
Nabestelnr. 00711621 / 00711042
Yeni Sipariş Numarası: 00711042
주문 번호 00711042

INSTRUCTIONS FOR USE

Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso
Instrucciones de uso
Instruções de Utilização
Brugsanvisning
Gebruiksaanwijzing
Kullanım talimatları
사용 설명서

*listening... and delivering solutions**

5976 Heisley Road
Mentor OH 44060

phone 440 / 639.4494

fax 440 / 639.4495

customer service 800 / 769.8226

www.usendoscopy.com

CE 0086

STERILE EO

Rx Only (U.S.A.)

INTENDED USE:

The Ponsky Endo-Sock™ retrieval pouch is designed to assist in the retrieval of infectious or suspicious tissue and gall stones during laparoscopic procedures.


WARNINGS AND PRECAUTIONS:

Laparoscopic procedures should only be performed by persons having adequate training and familiarity with laparoscopic techniques. Consult the medical literature relative to techniques, complications and hazards prior to the performance of any surgical laparoscopy procedure. If reference information for technique or contraindications is needed, please call US Endoscopy.

- Use of the Ponsky Endo-Sock™ retrieval pouch requires use of a 10mm or larger trocar sleeve.
- The Ponsky Endo-Sock™ retrieval pouch is packed in a sheath and should not be deployed prior to insertion. Premature deployment of the pouch from the sheath may damage the pouch and render the device inoperative. The pouch cannot be repackaged.
- The Ponsky Endo-Sock™ retrieval pouch must be inserted through the trocar sleeve as packaged.
- This disposable medical device is not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses this disposable medical device must bear full responsibility for its safety and effectiveness.

DIRECTIONS FOR USE:

- 1) Following recommended technique, prepare the patient for the laparoscopic surgical procedure.
- 2) Using sterile technique, remove the Ponsky Endo-Sock™ retrieval pouch from its package.

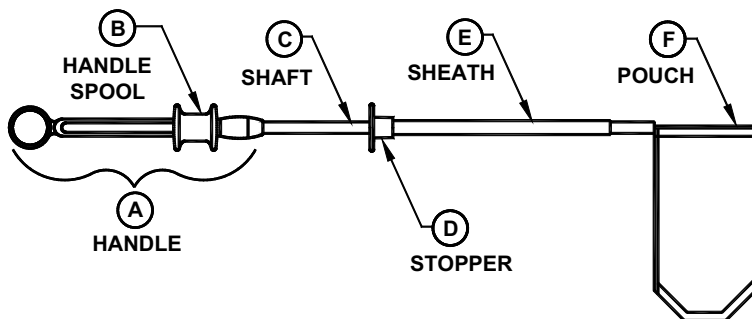- 3) To insert the Ponsky Endo-Sock™ retrieval pouch (DO NOT ACTIVATE HANDLE):
 - a) Grasp the shaft (C), just before the stopper (D), and insert the Ponsky Endo-Sock™ retrieval pouch sheath (E) through a 10mm or larger trocar sleeve.
 - b) Push the sheath (E) until the stopper (D) makes contact with the trocar head.
- 4) To deploy and open the pouch
 - a) Under direct visualization push the handle (A) forward to deploy the pouch.
 - b) To open the pouch, push the handle spool (B) forward.
- 5) Grasp tissue to be removed and place into the pouch. The Ponsky Endo-Sock™ retrieval pouch is torque stable and can be used to “scoop” tissue or stones.
- 6) Close the pouch by pulling back the handle spool (B). For the pouch to remain closed, tension must be maintained on the handle spool during the removal process.
- 7) To remove the pouch with tissue:
 - a) Pull the handle (A) back until the pouch makes contact with the distal end of the trocar sleeve.
 - b) Pull the trocar sleeve (with the sheath in it) out of the body until the pouch begins to exit from the port opening.
 - c) Grasp the pouch and determine the best method to remove the pouch with tissue. This may include removal of stones, morcellation of tissue, opening of port incision, etc.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: September 2009

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques de commerce détenues par US Endoscopy.
Listing...and delivering solutions®, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy
Listening...and delivering solutions®. US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, het ontwerp listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn geregistreerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.
Listening...and delivering solutions[®], US Endoscopy listening...and delivering solutions의 디자인과 [®] 또는 [™]이 표시된 모든 마크는 미국 특허청(the U.S. Patent and Trademark Office)에 등록되었거나 US Endoscopy 소유의 등록 상표입니다.