Velocity™ irrigation pump

Pompe d'irrigation Velocity™
Velocity™ Spülpumpe
Pompa di irrigazione Velocity™
Bomba de irrigación Velocity™
Bomba de irrigação Velocity™
Velocity™ udskylningspumpe
Velocity™ irrigatiepomp
Velocity™ irigasyon pompası
Velocity™ 세정 펌프

Reorder No. 00711520

Référence de commande 00711520
Nachbestell-Nr. 00711520
N. di riordino 00711520
N.º de pedido 00711520
N.º de encomenda 00711520
Efterbestillingsnummer. 00711520
Nabestelnr. 00711520
Yeni Sipariş Numarası: 00711520
주문 번호 00711520

Velocity™ irrigation pump

Pompe d'irrigation Velocity™
Velocity™ Spülpumpe
Pompa di irrigazione Velocity™
Bomba de irrigación Velocity™
Bomba de irrigação Velocity™
Velocity™ udskylningspumpe
Velocity™ irrigatiepomp
Velocity™ irigasyon pompası
Velocity™ 세정 펌프

Reorder No. 00711521

Référence de commande 00711521
Nachbestell-Nr. 00711521
N. di riordino 00711521
N.º de pedido 00711521
N.º de encomenda 00711521
Efterbestillingsnummer. 00711521
Nabestelnr. 00711521
Yeni Sipariş Numarası: 00711521
주문 번호 00711521

INSTRUCTIONS FOR USE

Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso
Instrucciones de uso
Instruções de Utilização
Brugsanvisning
Gebruiksaanwijzing
Kullanım talimatları
사용 설명서

listening... and delivering solutions®

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

Rx Only (U.S.A.)

Intended Use:

- The disposable Velocity™ irrigation pump is used in conjunction with a Velocity™ biopsy valve and a flexible gastrointestinal endoscope. This is a non-invasive device designed to facilitate the irrigation and lavage of the gastrointestinal tract.

Warnings and Precautions:

- This disposable medical device is intended to be used for no more than 24 hours after the package has been opened. This device should be discarded within this time period regardless of number of uses.
- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to complications, hazards and techniques prior to the performance of any endoscopic procedure.
- Use of the Velocity™ irrigation pump should not be attempted unless proficiency in technique has been developed by the clinician/endoscopist. Carefully monitor the amount of sterile water and air infused into the patient.
- Use proper techniques to avoid risk of aspiration.
- Performance may be compromised if the tubing is kinked and/or stepped on.
- Do not insert this device into the patient.
- The Velocity™ irrigation pump must only be used with US Endoscopy's Velocity™ biopsy valves (#00711139 / #00711140).
- Use acceptable medical techniques to avoid contaminating the Water Bottle Tubing and all luer lock connections during device setup and usage.
- Exposure to bodily fluids may occur during the connection or disconnection of this device. The User is responsible for adhering to Body Substance Isolation protocols.
- After each single patient use, visually inspect the Velocity™ biopsy valve (between the one-way valve and the Female Luer Lock Connector). If any contamination is suspected, discard the Velocity™ irrigation pump, Velocity™ biopsy valve and water bottle.
- US Endoscopy did not design this device to be reprocessed or used for more than the 24 hour period after the package has been opened, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- US Endoscopy does not take responsibility for the safety and effectiveness of this disposable medical device following reprocessing, remanufacturing, and/or reesterilization by any institution, practitioner, or third party or if it has been used for more than 24 hours.

Contraindications:

- Contraindications include those specific to any endoscopic procedure.

Instructions for Use:

Prior to clinical use, familiarize yourself with the device.

- Read these Instructions for Use and the Instructions for Use for the Velocity™ biopsy valve (#00711139 / #00711140).
- Inspect the package for shipping or handling damage. If damage is evident, do not use this device.
- Review the diagram (see Figure 1).

1. Setting up the Velocity™ irrigation pump:

- Open a bottle of sterile water (1000cc, or 1500cc). The Lid Cap on the Velocity™ irrigation pump will fit standard disposable sterile water bottles (e.g., Abbott or Baxter).
- Open the sterile pouch and remove the Velocity™ irrigation pump. Avoid contact with the Water Bottle Tubing and all luer lock connections to reduce the risk of contamination.
- Insert the Water Bottle Tubing into the sterile water bottle and firmly tighten the threaded Lid Cap to ensure a seal. Do not over tighten.
- Place the Foot Pump on the floor, ensuring that the luer lock connection does not come into contact with contaminants.
- Ensure that the device is discarded within 24 hours after the package has been opened. A label is provided with the device.

2. Priming the Velocity™ irrigation pump before use:

- Open the Pinch Clamp on the Velocity™ irrigation pump and depress the Foot Pump.
- Once depressed, close the Pinch Clamp on the Velocity™ irrigation pump and release the Foot Pump.
- Repeat steps 2.a & 2.b until the Velocity™ irrigation pump is completely primed and filled with sterile water.
- Prime the pump tubing into an appropriate container.
- Once primed, close the Pinch Clamp on the Velocity™ irrigation pump to prevent leakage.

3. Connecting the Velocity™ irrigation pump to the gastrointestinal endoscope:

- Attach the Velocity™ biopsy valve (#00711139 / #00711140) to the endoscope in accordance with the Instructions for Use for the Velocity™ biopsy valve.
- Connect the Male Luer Lock Connector on the Velocity™ irrigation pump to the Female Luer Lock Connector on the Velocity™ biopsy valve (#00711139 / #00711140).
- Ensure the Pinch Clamp on the Velocity™ biopsy valve is closed securely.

4. Intraoperative operation:

- Open the Pinch Clamp on the Velocity™ irrigation pump.
- To irrigate, open the Pinch Clamp on the Velocity™ biopsy valve and apply direct foot pressure to depress the Foot Pump.
 - Use either the ball of the foot or the heel to depress the Foot Pump.
 - The amount of foot pressure applied to the Foot Pump is directly related to the flow rate of the sterile water.
 - To refill the pump for additional irrigation, remove the ball of the foot or the heel from the Foot Pump to allow sterile water back into the Foot Pump.
 - Close the Pinch Clamp on the Velocity™ biopsy valve when irrigation is complete and prior to suctioning.
 - Carefully monitor the amount of sterile water infused into the patient.
- If the sterile water supply is depleted intraoperatively and priming is needed while the Velocity™ irrigation pump is attached to the endoscope:
 - Close the Pinch Clamp on the Velocity™ biopsy valve.

- Insert the Water Bottle Tubing into the replacement sterile water bottle and firmly tighten the threaded Lid Cap to ensure a seal. Do not over tighten. Avoid contact with the Water Bottle Tubing to reduce the risk of contamination.
 - Open the Pinch Clamp on the Velocity™ biopsy valve. Depress the suction button on the endoscope or depress the Foot Pump until the system is primed. Carefully monitor the amount of air infused into the patient.
 - Close the Pinch Clamp on the Velocity™ biopsy valve.
 - Continue with the procedure.
5. Interprocedural disassembly (for multiple patient use):
If using the Velocity™ irrigation pump for more than one patient use:
- a. Close the Pinch Clamps on the Velocity™ irrigation pump and on the Velocity™ biopsy valve.
 - b. Disconnect the Male Luer Lock Connector on the Velocity™ irrigation pump from the Female Luer Lock Connector on the Velocity™ biopsy valve (#00711139 / #00711140). Avoid contact with the Male Luer Lock connection on the Velocity™ irrigation pump to reduce the risk of contamination.
 - c. After each single patient use, visually inspect the Velocity™ biopsy valve (between the one way valve and the Female Luer Lock Connector). If contamination is suspected, discard the Velocity™ irrigation pump, Velocity™ biopsy valve and water bottle.
 - d. Secure the Male Luer Lock Connector end of the Velocity™ irrigation pump tubing. The Twist Clamp may be used for this purpose. A Protective Cap is provided with each Velocity™ biopsy valve (product #00711139 or #00711140). After completing a procedure, place the Protective Cap on the Male Luer Lock Connector to minimize the risk of cross contamination.
 - e. Disconnect the Velocity™ biopsy valve from the endoscope and discard in accordance with the Instructions for Use for the Velocity™ biopsy valve.
 - f. Repeat Steps 3 and 4 above as appropriate.
6. Device disposal:
- a. The Velocity™ biopsy valve must be discarded after each use.
 - b. The Velocity™ irrigation pump must be discarded 24 hours after the package has been opened, regardless of number of patient uses.
 - c. Prior to disposal, close the pinch clamps on the Velocity™ irrigation pump and on the Velocity™ biopsy valve and discard.
 - d. Disconnect the Velocity™ biopsy valve from the endoscope.

Setup Tips

1. Make sure the disposable sterile water bottle is stable to avoid tipping or spillage.
2. Set up the disposable sterile water bottle away from electrical connections.
3. Ensure that the disposable sterile water bottle contains sterile water at all times.
4. Prime the pump tubing into an appropriate container.
5. The Twist Clamp may be used during the procedure to attach the Velocity™ irrigation pump's tubing to the endoscope in order to reduce the weight of the tubing and enhance the feel of the endoscope.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

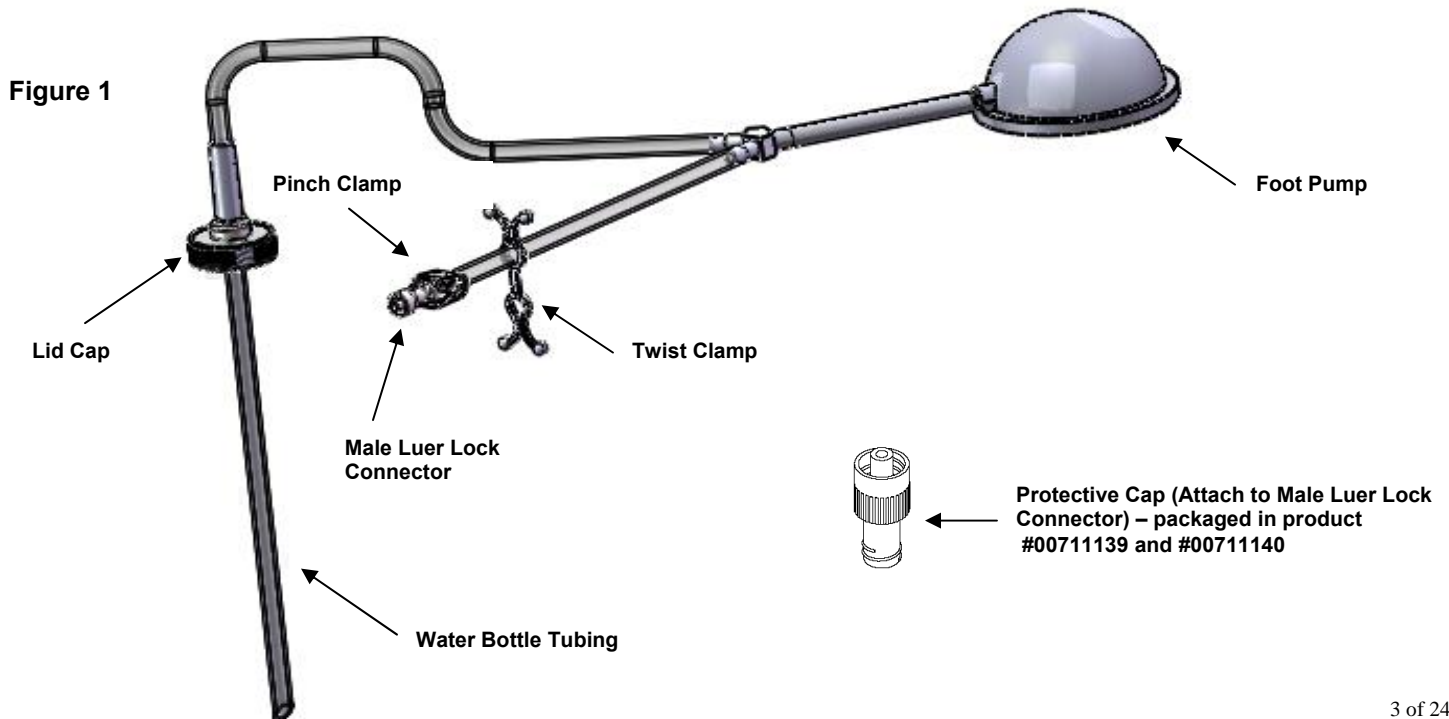
Issued Date: April 2010

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions® , das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, l'ideazione dello slogan listening...and delivering solutions di US Endoscopy e tutti i marchi contrassegnati da ® o ™ sono stati registrati presso l'US Patent and Trademark Office degli Stati Uniti o sono marchi di fabbrica di proprietà di US Endoscopy.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivery solutions®, het ontwerp listening...and delivery solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn gedeponeerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.