BioShield® – ERCP biopsy valve

Reorder No. 00711138

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INSTRUCTIONS FOR USE

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Brugsanvisning

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*listening...and delivering solutions**

5976 Heisley Road
Mentor, OH 44060 USA

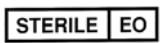
phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

CE 0086

RX Only (U.S.A.)

INTENDED USE:

The **BioShield**[®] – ERCP biopsy valve is used to cover the opening to the biopsy/suction channel of Olympus[®] and Fujinon[®] (G5 series and newer)[®] gastrointestinal endoscopes. The **BioShield**[®] – ERCP biopsy valve provides access for long-wire ERCP endoscopic device passage and exchange, helps maintain insufflation, and minimizes leakage of biomaterial from the biopsy port throughout the gastrointestinal endoscopic procedure.

WARNINGS AND PRECAUTIONS:

- Do not use a sharp or pointed object to prime the biopsy valve prior to use.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.
- The **BioShield**[®] – ERCP biopsy valve is not recommended for use with short-wire ERCP device systems.

DIRECTIONS FOR USE:

1. The **BioShield**[®] – ERCP biopsy valve is recommended for use with long-wire ERCP devices.
2. Securely place the biopsy valve onto the biopsy/suction channel opening of Olympus[®] and Fujinon[®] (G5 series and newer)[®] gastrointestinal endoscopes.
3. To achieve best valve performance, mount the biopsy valve in the recommended orientation below.

Long-wire ERCP devices

4. This biopsy valve does not accommodate a luer-lock syringe for irrigation purposes without an irrigating adaptor such as US Endoscopy part #00711131 (sold separately).

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: April 2010

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions[®], the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

Olympus[®] is a registered trademark of Olympus Optical Co., Ltd.

Fujinon[®] is a registered trademark of Fuji Film Co., Ltd.

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Listening...and delivering solutions[®], het ontwerp listening...and delivering solutions van US Endoscopy en alle aanduidingen met © of ™ zijn geregistreerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.
Listening...and delivering solutions[®], le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions[®], das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) registriert wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions[®], o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions[®], US Endoscopy listening...and delivering solutions design og alle varemærker markeret med [®] eller [™] er registreret i U.S. Patent and Trademark Office eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청 (U.S. Patent and Trademark Office) 에 등록되어 있거나 US Endoscopy 가 소유하고 있는 상표입니다.