BioGuard™ air/water & suction valves – Olympus
Clapets air-eau et clapets d'aspiration BioGuard™ – Olympus
BioGuard™ Luft-/Wasser- und Saugventile – Olympus
Valvole dell'aria/acqua e di aspirazione BioGuard™ – Olympus
Válvulas de aire/agua y succión BioGuard™ – Olympus
Válvulas de ar/água e de sucção BioGuard™ – Olympus
BioGuard™ luft/vand- og suge-ventiler – Olympus
BioGuard™ lucht-/water- en zuigkleppen – Olympus
BioGuard™ hava/su ve emme kapakları – Olympus
BioGuard™ 공기/물및 석션 밸브 – Olympus

Reorder No. 00711780

BioGuard™ air/water & suction valves – Olympus (3-pc kit)
Clapets air-eau et clapets d'aspiration BioGuard™ – Olympus (trousse, 3 pièces)
BioGuard™ Luft-/Wasser- und Saugventile – Olympus (3-teiliges Kit)
Valvole dell'aria/acqua e di aspirazione BioGuard™ – Olympus (Kit di 3 pz)
Válvulas de aire/agua y succión BioGuard™ – Olympus (kit de 3 piezas)
Válvulas de ar/água e de sucção BioGuard™ – Olympus (Kit de 3 peças)
BioGuard™ luft/vand- og suge-ventiler – Olympus (Kit med 3 stk.)
BioGuard™ lucht-/water- en zuigkleppen – Olympus (set met 3 stuks)
BioGuard™ hava/su ve emme kapakları – Olympus (3 parçalı set)
BioGuard™ 공기/물및 석션 밸브 – Olympus (3-PC 키트)

Reorder No. 00711783

BioGuard™ air/water & suction valves – Olympus (4-pc kit)
Clapets air-eau et clapets d'aspiration BioGuard™ – Olympus (trousse, 4 pièces)
BioGuard™ Luft-/Wasser- und Saugventile – Olympus (4-teiliges Kit)
Valvole dell'aria/acqua e di aspirazione BioGuard™ – Olympus (Kit di 4 pz)
Válvulas de aire/agua y succión BioGuard™ – Olympus (kit de 4 piezas)
Válvulas de ar/água e de sucção BioGuard™ – Olympus (Kit de 4 peças)
BioGuard™ luft/vand- og suge-ventiler – Olympus (Kit med 4 stk.)
BioGuard™ lucht-/water- en zuigkleppen – Olympus (set met 4 stuks)
BioGuard™ hava/su ve emme kapakları – Olympus (4 parçalı set)
BioGuard™ 공기/물및 석션 밸브 – Olympus (4-PC 키트)

Reorder No. 00711784

INSTRUCTIONS FOR USE

MODE D'EMPLOI

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BRUGSANVISNING

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KULLANIM TALİMATLARI

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*listening...and delivering solutions**

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

Not made with natural rubber latex.

Product Description	Product Number
BioGuard™ air/water & suction valves	00711780
BioGuard™ air/water & suction valves, BioShield® biopsy valve (3-pc kit)	00711783
BioGuard™ air/water & suction valves, BioShield® biopsy valve, Torrent® irrigation scope connector (4-pc kit)	00711784

Intended Use:

The single use **BioGuard™ air/water valve** is used to control the air / water function of an endoscope during GI endoscopic procedures.

The single use **BioGuard™ suction valve** is used to control the suction function of an endoscope during GI endoscopic procedures.

The single use **BioShield® biopsy valve** is used to cover the opening to the biopsy/suction channel of Olympus and G5 and newer Fujinon gastrointestinal endoscopes. The BioShield® biopsy valve provides access for endoscopic device passage and exchange, helps maintain insufflation, minimizes leakage of biomaterial from the biopsy port through the endoscopic procedure and provides access for irrigation.

The single use **Torrent® irrigation scope connector** is used in conjunction with the Torrent® irrigation tubing (tubing and accessories to accommodate various endoscopes and irrigation pumps) and are intended to provide irrigation via irrigation fluids such as sterile water supplied during gastrointestinal endoscopic procedures when used in conjunction with an irrigation pump (or electro-surgical unit).

Warnings and Precautions:

These devices are intended to be used with Olympus® GI Endoscopes **EXCLUDING Olympus® echoendoscopes that utilize a balloon channel**

(Follow the Olympus guidelines for air/water/suction use in the Olympus endoscopes)

- US Endoscopy did not design these devices to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of these devices to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, re-sterilizes, and/or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.
- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to techniques, contraindications, complications and hazards prior to any endoscopic procedure.
- When handling these devices use appropriate aseptic technique.

BioGuard™ air/water & suction valves:

- Make sure the holes in the air/water and suction valves are not clogged or blocked. (This could cause continuous air/CO₂ flow, or inadequate suction.)

BioShield® biopsy valve:

- Do not use a sharp or pointed object to prime the valve prior to use.
- Do not leave a device hanging from the valve. Doing so can cause the creation of a larger valve slit/hole that may cause leakage.
- If the lid of the valve is opened while attached to the endoscope during a procedure, scope suction will be compromised and leakage may occur.
- If leakage occurs, a sterile gauze should be used to cover the valve.

Torrent® irrigation scope connector:

- Never attach or reattach the Torrent® irrigation scope connector to an endoscope that is/was inside the patient and has not been reprocessed.
- Exposure to bodily fluids may occur during the connection or disconnection of this device. The User is responsible for adhering to Body Substance Isolation protocols.
- Do not insert this device into the patient. The device should be attached to the auxiliary water port of a flexible gastrointestinal endoscope and an irrigation source. Use proper techniques to avoid risk of aspiration of a patient.
- Always prime the auxiliary water channel or biopsy channel prior to inserting the endoscope into the patient.
- The Torrent irrigation scope connector is intended to be used only with the Torrent® irrigation tubing (00711560 / 00711561 / 00711562 / 00711563). The Torrent irrigation scope connector is not intended to be used with nor has it been tested with any other tubing.

Contraindications: Contraindications include those specific to any endoscopic procedure.

Directions for Use:

1. Prior to clinical use, familiarize yourself with the device and read all the Instructions for Use.
2. Inspect the package for shipping or handling damage. If damage is evident, **DO NOT USE THESE DEVICES, SAVE THEM FOR RETURN, AND CONTACT YOUR LOCAL PRODUCT SPECIALIST.**

BioGuard™ suction valve:

3. Remove the suction valve from the tray.
4. Insert the suction valve into suction cylinder of an Olympus endoscope (see Figure 1 below).
5. Twist valve slightly to seat in proper orientation within the cylinder.
6. Confirm that the valve fits securely and that there is no gap between the valve end cap and the endoscope.
Note: The valve should not be able to be rotated once it is fully inserted into the cylinder.
7. Actuate the valve by pressing and releasing a few times to confirm smooth function.

Note: Dipping in water may facilitate ease of use.

BioGuard™ air/water valve:

8. Remove the air/water valve from the tray.
9. Insert the air/water valve into the air/water cylinder of an Olympus endoscope (see Figure 1 below).
10. Confirm that the valve fits securely and that there is no gap between the valve end cap and the endoscope.
11. Actuate the valve by pressing and releasing a few times to confirm smooth function.

Note: Dipping in water may facilitate ease of use.

BioShield® biopsy valve:

12. If using the 3 piece kit (See Figure 2), attach the **BioShield® biopsy valve**.
13. Securely place the biopsy valve onto the biopsy/suction channel opening of an Olympus endoscope.
Note: This biopsy valve does not accommodate a luer-lock syringe without an irrigating adaptor (part #00711131; sold separately).

Torrent® irrigation scope connector:

14. If using the 4 piece kit (See Figure 3), attach the **Torrent® irrigation scope connector**.
15. Attach the Torrent® irrigation scope connector to the Torrent® irrigation tubing (00711560 / 00711561 / 00711562 / 00711563). For use of the Torrent® irrigation tubing, reference Instructions for Use.
16. Attach the Torrent® irrigation scope connector to the auxiliary water port by screwing it onto the port. **When attaching the scope connector to the port, take care to ensure that the scope connector is aligned to the port and avoid excessive tightening.** Misalignment or excessive tightening of the scope connector could lead to the possibility of leakage.
17. Once the scope connector is attached to the auxiliary water port of the endoscope, attach Torrent® irrigation tubing to the Torrent® irrigation scope connector. The Torrent® irrigation scope connector is manufactured with a one-way valve to minimize the risk of cross-contamination of the Torrent® irrigation system.
18. Place distal tip of endoscope into water filled receptacle.
19. Press and hold the suction valve for a few seconds and then release.
20. Confirm that endoscope is suctioning water.
21. With distal tip of scope still under water, rest finger over hole in air/water valve.
22. Confirm that air is exiting the tip of scope, which is evident by the production of air bubbles.
23. Remove distal tip of endoscope from water filled receptacle.
24. Press the air/water valve, and hold for 5-10 seconds.
25. Confirm that water exits the air/water nozzle on the end of the endoscope.
26. **Remove and discard the BioGuard™ air/water suction valves, the BioShield® biopsy valve, and the Torrent® irrigation scope connector after each patient use.**

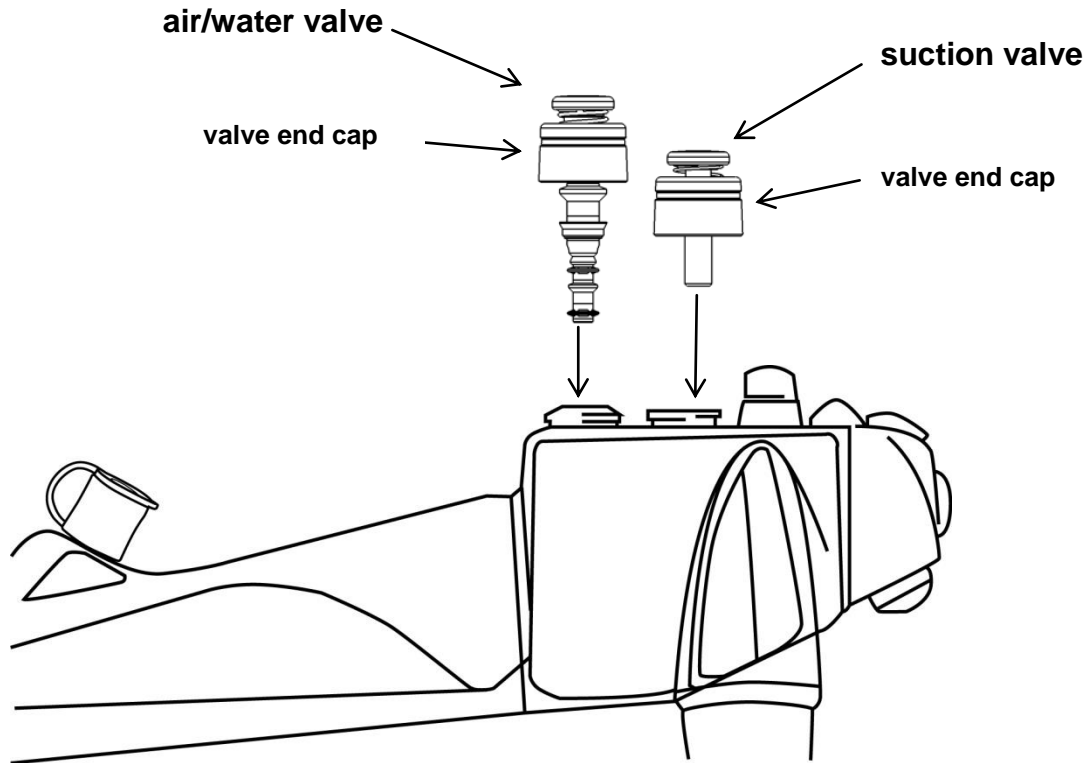
Figure 1

3-Piece Kit

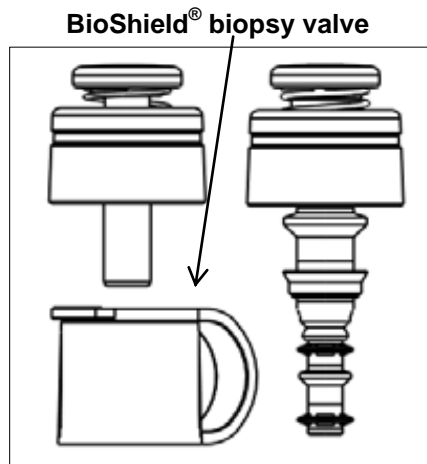
Figure 2

4-Piece Kit

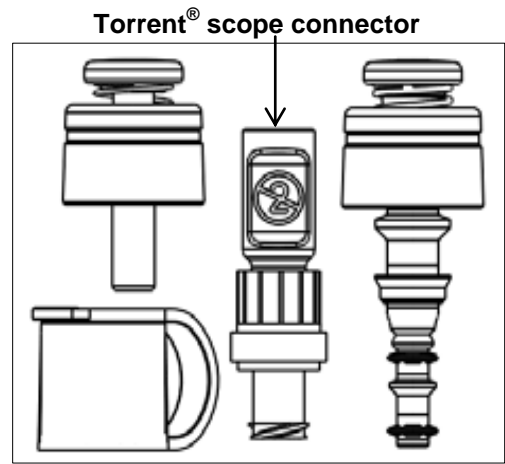
Figure 3

Product Disposal:

After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: March 2016

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

US Endoscopy, a wholly owned subsidiary of STERIS Corporation.
Olympus® is a registered trademark of Olympus corporation.

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Listening...and delivering solutions®, het ontwerp Listening...and delivering solutions van US Endoscopy en alle aanduidingen met © of ™ zijn gedeponeerd bij het Amerikaanse Octrooibureau of zijn handelsmerken van US Endoscopy.
Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont des marques déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listing...and delivering solutions®, das US Endoscopy ...and delivering solutions-Design und alle mit ® oder ™ gekennzeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, designet US Endoscopy listening...and delivering solutions og alle varemærker markeret med ® eller ™ er registreret i USA's patent- og varemærkeregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 USEndoscopy 가 소유하고 있는 상표입니다.